

REF	501-3 6x5mL	EXP	2027-06-11	LOT	2FCF5B6	更新日期: 2024-09 1 / 6
------------	----------------	------------	------------	------------	---------	------------------------

本报告含以下项目的示范实验室数据

参数 (Parameters)	参数 (Parameters)	参数 (Parameters)
(Li)锂	(CRBM)卡马西平	(DGXN)地高辛
(PHYT)苯妥英	(THEO)茶碱	(VALP)丙戊酸
(TT3)总三碘甲状腺氨酸	(FT3)游离T3	(TT4)总T4
(FT4)游离T4	(TSH)促甲状腺素	(Cortisol)皮质醇
(FSH)促卵泡刺激素	(LH)促黄体生成素	(PRL)催/泌乳素
(TES)睾酮	(E2)雌二醇	(C-P)C-肽
(FA)叶酸	(INS)胰岛素	(VB12)维生素B12
(IgG)免疫球蛋白 IgG	(IgM)免疫球蛋白 IgM	(IgA)免疫球蛋白 IgA
(IgE)免疫球蛋白 IgE	(AFP)甲胎蛋白	(CEA)癌胚抗原
(hCG)人绒毛膜促性腺激素	(PSA)前列腺特异性抗原	(Ferritin)铁蛋白
(F-PSA)游离前列腺特异性抗原	(FE3/uE3)游离雌三醇	(17-OHP)17-羟孕酮
(ASD)雄烯二酮	(ALD)醛固酮	(DHEA-S)硫酸脱氢表雄酮
(HGH)人生长激素	(25-OH-Vit D)25羟基维生素D	(ACTH)促肾上腺皮质激素
(AK)阿米卡星	(Gentamicin)庆大霉素	(Phenobarbital)苯巴比妥
(Tobramycin)妥布霉素	(Vancomycin)万古霉素	(CT)降钙素
(Acetaminophen)对乙酰氨基酚	(cPSA)复合前列腺特异性抗原	(Salicylate)水杨酸盐

REF 501-3	6x5mL	EXP 2027-06-11	LOT 2FCF5B6	更新日期: 2024-09
				2 / 6

批号: 2FCF5B6			
项目\仪器\试剂\方法	单位	均值	+ / - 2 SD
(17-OHP)17-羟孕酮			
maglumi system Snibe \ Chemiluminescence Immunoassay (CLIA)	ng/mL	6.49	5.19-7.79
(25-OH-Vit D)25羟基维生素D			
YHLO iFlash Analyzer YHL \ Chemiluminescence Immunoassay (CLIA)	ng/mL	21	16.8-25.2
YHLO iFlash Analyzer YHL \ Chemiluminescence Immunoassay (CLIA)	nmol/L	52.5	42-63
(Acetaminophen)对乙酰氨基酚			
Abbott Alinity C Series Abbott \ Colorimetry	mg/L	106	85-127
(ACTH)促肾上腺皮质激素			
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	pg/mL	216	173-259
(AFP)甲胎蛋白			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	262	210-314
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	244	195-293
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	258	206-310
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	294	235-353
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	275	220-330
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	249	199-299
(AK)阿米卡星			
Abbott Alinity C Series Abbott \ Immunoturbidimetry (ITA)	mg/L	25	20-30
(ALD)醛固酮			
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	pg/mL	588	470-706
(ASD)雄烯二酮			
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	3.2	2.56-3.84
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	2.34	1.88-2.8
(CEA)癌胚抗原			
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	38.6	30.9-46.3
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	35.7	28.6-42.8
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	36.9	29.5-44.3
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	39.7	31.8-47.6
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	33	26.4-39.6
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	35.6	28.5-42.7
(Cortisol)皮质醇			
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	283	226-340
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	269	215-323
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	263	210-316
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	233	186-280
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	292	234-350
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	328	262-394
(C-P)C-肽			
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	9.03	7.22-10.84
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	6.44	5.16-7.72
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	9.74	7.79-11.69
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	8.63	6.9-10.36
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	8.02	6.42-9.62

REF	501-3	6x5mL	EXP	2027-06-11	LOT	2FCF5B6	更新日期:	2024-09
								3 / 6

(cPSA)复合前列腺特异性抗原			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/L	5160	4118-6202
(CRBM)卡马西平			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	µg/mL	12.4	9.9-14.9
(CT)降钙素			
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/L	398	318-478
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/L	389	311-467
(DGXN)地高辛			
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	3.51	2.81-4.21
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	3.51	2.81-4.21
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	3.68	2.94-4.42
Abbott Alinity C Series Abbott \ Immunoturbidimetry (ITA)	ng/mL	3.48	2.78-4.18
(DHEA-S)硫酸脱氢表雄酮			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	4890	3912-5868
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	5530	4424-6636
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	6010	4808-7212
(E2)雌二醇			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	pmol/L	2397	1918-2876
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	pmol/L	1343	1074-1612
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	pmol/L	2341	1873-2809
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	pmol/L	1982	1586-2378
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	pmol/L	1204	961-1447
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	pmol/L	1185	948-1422
(FA)叶酸			
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	17.9	14.3-21.5
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	15.6	12.5-18.7
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	16.8	13.4-20.2
(FE3/uE3)游离雌三醇			
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	14.6	11.7-17.5
(Ferritin)铁蛋白			
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	462	370-554
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	345	276-414
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	706	565-847
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	580	464-696
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	361	289-433
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	418	334-502
(F-PSA)游离前列腺特异性抗原			
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	12	9.6-14.4
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	9.75	7.78-11.72
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	10.6	8.5-12.7
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	13	10.4-15.6
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	14.5	11.6-17.4
(FSH)促卵泡刺激素			
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	IU/L	48.9	39.1-58.7

REF	501-3	6x5mL	EXP	2027-06-11	LOT	2FCF5B6	更新日期:	2024-09
								4 / 6

Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	IU/L	46.2	37-55.4
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	IU/L	78.8	63-94.6
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	IU/L	48.9	39.1-58.7
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	IU/L	40.3	32.2-48.4
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	IU/L	58	46.4-69.6
(FT3)游离T3			
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	pmol/L	18.3	14.6-22
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	pmol/L	15	12-18
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	pmol/L	28	22.4-33.6
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	pmol/L	23.1	18.5-27.7
(FT4)游离T4			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	pmol/L	37.3	29.6-45
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	pmol/L	46.6	37.4-55.8
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	pmol/L	45.9	36.6-55.2
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	pmol/L	194	155-233
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	pmol/L	66.4	53-79.8
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	pmol/L	47.2	37.7-56.7
(Gentamicin)庆大霉素			
Abbott Alinity C Series Abbott \ Immunoturbidimetry (ITA)	nmol/L	13334	10667-16001
Siemens ADVIA Centaur CP Siemens \ Dipstick	nmol/L	12185	9772-14598
(hCG)人绒毛膜促性腺激素			
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	IU/L	201	161-241
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	IU/L	216	173-259
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	IU/L	211	169-253
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	IU/L	223	178-268
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	IU/L	213	170-256
(HGH)人生长激素			
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	µg/L	16.6	13.3-19.9
(IgA)免疫球蛋白 IgA			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	mg/dL	170	136-204
(IgE)免疫球蛋白 IgE			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	IU/mL	101	81-121
(IgG)免疫球蛋白 IgG			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	mg/dL	687	550-824
(IgM)免疫球蛋白 IgM			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	mg/dL	65.4	52.3-78.5
(INS)胰岛素			
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	mIU/L	154	123-185
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	mIU/L	185	148-222
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	mIU/L	149	119-179
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	mIU/L	212	170-254
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	mIU/L	219	175-263
(LH)促黄体生成素			
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	IU/L	102	82-122
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	IU/L	47.9	38.3-57.5
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	IU/L	57.3	45.8-68.8

REF	501-3	6x5mL	EXP	2027-06-11	LOT	2FCF5B6	更新日期:	2024-09
								5 / 6

Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	IU/L	74.6	59.7-89.5
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	IU/L	57.3	45.8-68.8
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	IU/L	58.2	46.6-69.8
(Li) 锂			
Abbott Alinity C Series Abbott \ ISE (direct)	mmol/L	1.78	1.42-2.14
(Phenobarbital) 苯巴比妥			
Abbott Alinity C Series Abbott \ Immunoturbidimetry (ITA)	nmol/L	1862	1490-2234
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	nmol/L	284483	227586-341380
(PHYT) 苯妥英			
Abbott Alinity C Series Abbott \ Enzyme immunoassay (EIA)	µg/mL	23.3	18.6-28
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	µg/mL	29.6	23.7-35.5
(PRL) 催乳素			
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	36.9	29.5-44.3
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	36.2	29-43.4
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	48.1	38.5-57.7
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	35.2	28.2-42.2
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	50.5	40.4-60.6
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	53.3	42.6-64
(PSA) 前列腺特异性抗原			
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	20.8	16.6-25
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	17.8	14.2-21.4
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	17.5	14-21
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	19.3	15.4-23.2
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	3.18	2.54-3.82
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	17.7	14.2-21.2
(Salicylate) 水杨酸盐			
Abbott Alinity C Series Abbott \ Enzymatic	mg/dL	43	34.4-51.6
(TES) 睾酮			
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	0.007	0.006-0.008
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	4.21	3.37-5.05
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	7.35	5.89-8.81
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	8.21	6.57-9.85
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	7.35	5.88-8.82
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	12	9.6-14.4
(THEO) 茶碱			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	µg/mL	24.4	19.5-29.3
Abbott Alinity C Series Abbott \ Enzyme immunoassay (EIA)	µg/mL	25.1	20.1-30.1
(Tobramycin) 妥布霉素			
Siemens ADVIA Centaur CP Siemens \ Dipstick	nmol/L	11499	9176-13822
Abbott Alinity C Series Abbott \ Immunoturbidimetry (ITA)	nmol/L	10407	8346-12468
(TSH) 促甲状腺素			
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	mIU/L	37	29.6-44.4
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	mIU/L	36.3	29-43.6
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	mIU/L	36.7	29.4-44
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	mIU/L	29.8	23.8-35.8
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	mIU/L	26	20.8-31.2

REF	501-3	6x5mL	EXP	2027-06-11	LOT	2FCF5B6	更新日期:	2024-09
								6 / 6

Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	miU/L	30.3	24.2-36.4
(TT3)总三碘甲状腺原氨酸			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	4.04	3.24-4.84
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	3.33	2.66-4
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	3.29	2.63-3.95
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	3.48	2.78-4.18
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	2.84	2.28-3.4
(TT4)总T4			
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	152	122-182
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA)	ng/mL	94.3	75.4-113.2
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	176	141-211
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	155	124-186
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	152	122-182
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	ng/mL	147	118-176
(VALP)丙戊酸			
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	µg/mL	123	98-148
Abbott Alinity C Series Abbott \ Immunoturbidimetry (ITA)	µg/mL	122	98-146
(Vancomycin)万古霉素			
Abbott Alinity C Series Abbott \ Immunoturbidimetry (ITA)	nmol/L	23115	18492-27738
Siemens ADVIA Centaur CP Siemens \ Dipstick	nmol/L	17595	14076-21114
(VB12)维生素B12			
Abbott Alinity i Series Abbott \ Chemiluminescence Immunoassay (CLIA)	pg/mL	791	633-949
Mindray CL-2000i/1000i/6000i series Mindray \ Chemiluminescence Immunoassay (CLIA)	pg/mL	876	701-1051
Siemens ADVIA Centaur CP Siemens \ Chemiluminescence Immunoassay (CLIA)	pg/mL	808	646-970
Beckman Access Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	pg/mL	625	500-750
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	pg/mL	612	490-734

注: 此报告所提供的数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来, 仅供学习、参考之用。因所用技术、仪器和试剂的不同, 或因制造商检测方法的改变, 均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求, 实验室须遵循相关技术规范确立自己的均值和可接受范围。