

REF	671-2	6×1mL	EXP	2023-08-25	LOT	40033B6	更新日期：	2024-04
1 / 3								

		批号：40033B6			
项目\仪器\试剂\方法	单位	均值 \ CV		Lab \ 点数	
(APTT)活化部分凝血活酶时间					
OTHER	Seconds	90.6	13.5%	9	201
Mindray \ Magnetic Beads	Seconds	73.2	9%	8	113
Werfen ACL series	Seconds	67.5	6.7%	77	2831
SekiSui \ OTHER	Seconds	50.4	4.3%	71	1592
Sysmex CS 5100	Seconds	53.4	7.7%	6	147
Dedicated \ Light scattering	Seconds	89.7	10.7%	<5	<100
Werfen ACL series	Seconds	67.2	5.5%	26	614
Werfen \ OTHER	Seconds	102	8.3%	17	314
BE Thrombolyzer X/XRC/XRM	Seconds	52.5	4.4%	14	317
Dedicated \ Light scattering	Seconds	57.3	7.9%	13	295
Mindray EXC810	Seconds	55.2	5.5%	13	276
Mindray \ OTHER	Seconds	72.3	4%	12	316
Sysmex CS 5100	Seconds	51	2.6%	12	265
sysmex \ OTHER	Seconds	81.4	12%	<5	<100
Sysmex CS-1300	Seconds	88.9	9.8%	<5	<100
sysmex \ Light scattering	Seconds	58.6	10.1%	<5	<100
Beckman AU series	Seconds	102	8.8%	<5	<100
Dedicated \ Light scattering	Seconds	59.1	12%	<5	<100
Sekisui Coapresta 3000	Seconds				
SekiSui \ OTHER	Seconds				
OTHER	Seconds				
OTHER \ OTHER	Seconds				
Sysmex CS-2400	Seconds				
sysmex \ OTHER	Seconds				
OTHER	Seconds				
Dedicated \ OTHER	Seconds				
Stago STA series	Seconds				
Stago \ Magnetic Beads	Seconds				
Sysmex CS 5100	Seconds				
Siemens \ Light scattering	Seconds				
Werfen ACL series	Seconds				
Werfen \ Light scattering	Seconds				
Sysmex CA 1500/6000/7000	Seconds				
Siemens \ Light scattering	Seconds				
Sekisui Coapresta 3000	Seconds				
SekiSui \ Light scattering	Seconds				
(AT-III)抗凝血酶III					
OTHER	%	39.5	13.6%	9	204
Mindray \ OTHER	%	45.1	6.2%	8	<100
Sysmex CS-2400	%	68	7.2%	8	100
sysmex \ OTHER	%	45.6	10%	70	1527
Werfen ACL series	%	58.8	14.1%	6	146
SekiSui \ OTHER	%	41	11.9%	26	545
Werfen ACL series	%	46.1	9.2%	20	466
Werfen \ OTHER	%	49	9.8%	<5	<100
BE Thrombolyzer X/XRC/XRM	%	37.1	16%	13	331
Dedicated \ Magnetic Beads	%	40	12%	<5	<100
Beckman AU series	%	42.9	9.8%	<5	<100
Dedicated \ Nephelometry	%				
Sysmex CS 5100	%				
Dedicated \ Nephelometry	%				
Sekisui Coapresta 3000	%				
SekiSui \ Nephelometry	%				
Sekisui Coapresta 3000	%				
SekiSui \ OTHER	%				
Stago STA series	%				
Stago \ Magnetic Beads	%				
Werfen ACL series	%				
Werfen \ Nephelometry	%				
(D-D(DDU)) D-二聚体(DDU)					
OTHER	mg/L	3.52	24%	9	183
Mindray \ OTHER	mg/L	3.88	9.9%	8	140
Sekisui Coapresta 3000	mg/L	2.96	6.4%	8	124
SekiSui \ OTHER	mg/L	0.896	18%	65	1572
Werfen ACL series	mg/L	3.82	9.4%	6	327
SekiSui \ OTHER	mg/L	1.74	17.9%	<5	<100
Werfen ACL series	mg/L	3.28	12%	29	929
Werfen \ OTHER	mg/L	2.33	32.7%	26	588
Sysmex CS 2000i/2100i	mg/L	0.74	6.8%	25	541
Siemens \ Nephelometry	mg/L	1.73	11.2%	18	465
Mindray EXC810	mg/L	3.86	5.4%	18	422
Mindray \ OTHER	mg/L	3.36	8.2%	12	309
Sysmex CS 5100	mg/L	0.81	7.2%	11	237
Dedicated \ Nephelometry	mg/L	3.99	12%	<5	<100
Sysmex CS 5100	mg/L	3.88	12%	<5	<100
sysmex \ OTHER	mg/L				
OTHER	mg/L				
OTHER \ OTHER	mg/L				
Beckman AU series	mg/L				
Dedicated \ Nephelometry	mg/L				
Sysmex CS 5100	mg/L				
Siemens \ Nephelometry	mg/L				
Sysmex CS-2400	mg/L				
Siemens \ Nephelometry	mg/L				
OTHER	mg/L				
Beckman Coulter \ Latex agglutination (LA)	mg/L				
Sysmex CS 5100	mg/L				
SekiSui \ Nephelometry	mg/L				
Sekisui Coapresta 3000	mg/L				
SekiSui \ Nephelometry	mg/L				

REF	671-2	6×1mL	EXP	2023-08-25	LOT	40033B6	更新日期：	2024-04
2 / 3								

		批号：40033B6			
项目\仪器\试剂\方法	单位	均值 \ CV		Lab \ 点数	
Werfen ACL series Werfen \ Nephelometry	mg/L	0.756	9.7%	<5	<100
(D-D(FEU)) D-二聚体(FEU)					
BE Thrombolyzer X/XRC/XRM Dedicated \ OTHER	mg/L	0.941	3.4%	6	146
Sysmex CS 5100 Dedicated \ Nephelometry	mg/L	2.14	87.6%	45	1770
Sysmex CS-1300 sysmex \ Nephelometry	mg/L	3.3	12.9%	18	415
Sysmex CS 5100 sysmex \ Nephelometry	mg/L	2.86	10.1%	<5	<100
Stago STA series Stago \ Latex agglutination (LA)	mg/L	1.63	8.6%	<5	<100
(FDP)纤维蛋白(原)降解产物					
OTHER Mindray \ OTHER	mg/L	9.29	8.9%	9	184
Werfen ACL series SekiSui \ OTHER	mg/L	4.15	7.1%	8	106
Sekisui Coapresta 3000 SekiSui \ OTHER	mg/L	6.25	5.1%	7	133
BE Thrombolyzer X/XRC/XRM Dedicated \ Magnetic Beads	mg/L	4.66	16.2%	6	147
Werfen ACL series Werfen \ OTHER	mg/L	5.71	13%	34	833
Sysmex CS 5100 Dedicated \ Nephelometry	mg/L	10.8	20%	28	555
Sysmex CS-1300 sysmex \ Nephelometry	mg/L	10.6	15.1%	17	354
Werfen ACL series SekiSui \ Nephelometry	mg/L	5.33	9.2%	<5	<100
Sysmex CS 5100 SekiSui \ Nephelometry	mg/L	6.9	12%	<5	<100
Stago STA series Stago \ Nephelometry	mg/L	5.56	8.8%	<5	<100
(FIB)纤维蛋白原					
OTHER Mindray \ Magnetic Beads	g/L	1.96	5.4%	9	165
Werfen ACL series Werfen \ OTHER	g/L	2.09	9.3%	78	1728
Sysmex CS 5100 Dedicated \ Light scattering	g/L	1.96	5.9%	77	2816
BE Thrombolyzer X/XRC/XRM Dedicated \ Light scattering	g/L	2.16	8%	6	146
Beckman AU series Dedicated \ Light scattering	g/L	2.11	7.5%	26	659
Sysmex CS 5100 sysmex \ OTHER	g/L	1.92	6.8%	26	598
Mindray EXC810 Mindray \ OTHER	g/L	1.98	8.3%	<5	<100
Sysmex CS-1300 sysmex \ Light scattering	g/L	1.78	16.8%	18	413
OTHER OTHER \ OTHER	g/L	2.16	7.5%	18	247
Sekisui Coapresta 3000 SekiSui \ OTHER	g/L	2.09	8.7%	13	310
Sysmex CS-2400 sysmex \ OTHER	g/L	1.87	6.3%	12	316
OTHER Dedicated \ OTHER	g/L	2.14	6%	12	261
OTHER Beckman Coulter \ Magnetic Beads	g/L	2.24	6%	12	236
Sysmex CA 1500/6000/7000 Siemens \ Light scattering	g/L	2.19	9.6%	<5	<100
Stago STA series Stago \ Magnetic Beads	g/L	2.15	12%	<5	<100
Werfen ACL series Werfen \ Light scattering	g/L	2.38	9.7%	<5	<100
Sekisui Coapresta 3000 SekiSui \ Light scattering	g/L	1.91	12%	<5	<100
Sysmex CS 5100 Siemens \ Light scattering	g/L	1.72	9.3%	<5	<100
(PT INR)PT国际标准化比值					
OTHER Mindray \ Calculate	INR	2.11	7.1%	9	159
Sysmex CS 5100 Dedicated \ Calculate	INR	2.74	5.1%	72	2783
BE Thrombolyzer X/XRC/XRM Dedicated \ Calculate	INR	2.76	6.6%	6	139
Werfen ACL series Werfen \ Calculate	INR	2.72	5.4%	37	781
Mindray EXC810 Mindray \ Calculate	INR	2.02	8%	<5	<100
Beckman AU series Dedicated \ Calculate	INR	2.68	6.4%	26	639
Sysmex CS 5100 sysmex \ OTHER	INR	2.67	5.5%	26	612
Sysmex CS-1300 sysmex \ Calculate	INR	2.39	12.9%	17	311
Sekisui Coapresta 3000 SekiSui \ Calculate	INR	2.79	6.9%	14	345

REF	671-2	6×1mL	EXP	2023-08-25	LOT	40033B6	更新日期：	2024-04
3 / 3								

		批号：40033B6			
项目\仪器\试剂\方法	单位	均值 \ CV		Lab \ 点数	
Sysmex CS-2400 sysmex \ Calculate	INR	2.62	7.2%	12	320
OTHER OTHER \ OTHER	INR	2.79	4.7%	12	264
OTHER OTHER \ Calculate	INR	2.86	4.9%	12	237
Werfen ACL series SekiSui \ Calculate	INR	2.55	4.3%	10	116
Stago STA series Stago \ Calculate	INR	2.75	12%	<5	<100
(PT)血浆凝血酶原时间					
OTHER Mindray \ Magnetic Beads	Seconds	25.7	7.2%	9	160
Sysmex CS 5100 Dedicated \ Light scattering	Seconds	30.7	4.8%	75	2799
Werfen ACL series Werfen \ OTHER	Seconds	29.7	6.4%	70	1621
BE Thrombolyzer X/XRC/XRM Dedicated \ Magnetic Beads	Seconds	31.7	6.4%	5	124
Mindray EXC810 Mindray \ OTHER	Seconds	24.7	7.9%	<5	<100
Sysmex CS 5100 sysmex \ OTHER	Seconds	30.1	5%	26	608
Beckman AU series Dedicated \ Light scattering	Seconds	29.5	6.9%	23	493
Sysmex CS-1300 sysmex \ Light scattering	Seconds	27.1	11%	18	350
Sekisui Coapresta 3000 SekiSui \ OTHER	Seconds	33.9	5%	16	307
OTHER OTHER \ OTHER	Seconds	33.5	4.7%	15	259
Sysmex CS-2400 sysmex \ OTHER	Seconds	28.2	5.1%	12	319
OTHER Dedicated \ OTHER	Seconds	31.3	4.5%	12	262
Werfen ACL series SekiSui \ OTHER	Seconds	28.1	4.3%	10	117
Stago STA series Stago \ Magnetic Beads	Seconds	29	12%	<5	<100
Werfen ACL series Werfen \ Light scattering	Seconds	31.2	10.3%	<5	<100
Sysmex CA 1500/6000/7000 Siemens \ Light scattering	Seconds	33.4	9.6%	<5	<100
Sysmex CS 5100 Siemens \ Light scattering	Seconds	34.9	9.2%	<5	<100
Sekisui Coapresta 3000 SekiSui \ Light scattering	Seconds	28.7	12%	<5	<100
Sysmex CS 5100 Dedicated \ Magnetic Beads	Seconds	28.2	0%	<5	<100
(TT)凝血酶时间					
OTHER Mindray \ Magnetic Beads	Seconds	19	5.9%	9	172
Werfen ACL series Werfen \ OTHER	Seconds	18.4	6.6%	78	1695
Sysmex CS 5100 Dedicated \ Light scattering	Seconds	17.2	8.2%	77	2829
BE Thrombolyzer X/XRC/XRM Dedicated \ Light scattering	Seconds	10.5	3.2%	6	147
Mindray EXC810 Mindray \ OTHER	Seconds	17.4	8.9%	<5	<100
Sysmex CS 5100 sysmex \ OTHER	Seconds	17.4	4.3%	26	612
Beckman AU series Dedicated \ Light scattering	Seconds	16.8	3.5%	19	459
Sysmex CS-1300 sysmex \ Light scattering	Seconds	15.9	19.6%	18	421
OTHER OTHER \ OTHER	Seconds	20.5	7.7%	17	244
Sysmex CS-2400 sysmex \ OTHER	Seconds	18	5.2%	12	314
Sekisui Coapresta 3000 SekiSui \ OTHER	Seconds	20.6	2.9%	12	263
OTHER Dedicated \ OTHER	Seconds	18.8	5.5%	11	266
OTHER Beckman Coulter \ Magnetic Beads	Seconds	18.9	6.2%	11	234
Stago STA series Stago \ Magnetic Beads	Seconds	14.8	12%	<5	<100
Sekisui Coapresta 3000 SekiSui \ Light scattering	Seconds	17.8	12%	<5	<100
Sysmex CA 1500/6000/7000 sysmex \ Light scattering	Seconds	17.2	9.9%	<5	<100
Werfen ACL series SUNBIO \ Light scattering	Seconds	17.3	9.2%	<5	<100

注：此报告所提供的数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来，仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。