



REF	671-1	6×1mL	EXP	2025-09-03	LOT	4000D6F	更新日期：	2023-04
1 / 2								

本报告含以下项目的示范实验室数据

参数 (Parameters)	参数 (Parameters)	参数 (Parameters)
(APTT)活化部分凝血活酶时间	(AT-III)抗凝血酶Ⅲ	(D-D(DDU)) D-二聚体(DDU)
(D-D(FEU)) D-二聚体(FEU)	(FIB)纤维蛋白原	(PT)血浆凝血酶原时间
(TT)凝血酶时间		

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。

REF	671-1	6×1mL	EXP	2025-09-03	LOT	4000D6F	更新日期： 2023-04
							2 / 2

批号：4000D6F			
项目\仪器\试剂方法	单位	均值	+ / - 2 SD
(APTT)活化部分凝血活酶时间			
Werfen ACL series			
Werfen Hemosil SynthAsil APTT \ Light scattering	Seconds	32	25.6-38.4
Werfen ACL series			
Werfen Hemosil APTT-SP \ Light scattering	Seconds	28	22.4-33.6
Stago STA series			
Stago \ Magnetic Beads	Seconds	33	26.4-39.6
Sysmex CA 1500/6000/7000			
Siemens Dade Actin FS(APTT) \ Light scattering	Seconds	33	26.4-39.6
Sysmex CA 1500/6000/7000			
Siemens Dade Actin(APTT) \ Light scattering	Seconds	34	27.2-40.8
Sysmex CA 1500/6000/7000			
Siemens Pathrombin SL(APTT) \ Light scattering	Seconds	37	29.6-44.4
(AT-III)抗凝血酶III			
Werfen ACL series			
Werfen Hemosil Liquid Antithrombin (AT□) \ Nephelometry	%	96	76.8-115
Werfen ACL series			
Werfen IL Antithrombin III Chromogenic (AT□) \ Nephelometry	%	96	76.8-115
Sysmex CA 1500/6000/7000			
Siemens Innovance Antithrombin (AT□) \ Nephelometry	%	98	78.4-118
Sysmex CA 1500/6000/7000			
Siemens Berichrom Antithrombin III (AT□) \ Nephelometry	%	95	76-114
(D-D(DDU)) D-二聚体(DDU)			
Werfen ACL series			
Werfen Hemosil D-Dimer HS \ Nephelometry	mg/L	0.452	0.362-0.542
Werfen ACL series			
Werfen Hemosil D-Dimer \ Nephelometry	mg/L	0.472	0.378-0.566
(D-D(FEU)) D-二聚体(FEU)			
Stago STA series			
Stago \ Nephelometry	mg/L	0.77	0.61-0.93
Sysmex CA 1500/6000/7000			
Siemens \ Nephelometry	mg/L	1.36	1.09-1.63
(FIB)纤维蛋白原			
Werfen ACL series			
Werfen Hemosil QFA Thrombin (FIB) \ Light scattering	g/L	3.48	2.78-4.18
Werfen ACL series			
Werfen Hemosil Fib-C \ Light scattering	g/L	3.37	2.7-4.04
Stago STA series			
Stago \ Magnetic Beads	g/L	3.63	2.9-4.36
Sysmex CA 1500/6000/7000			
Siemens Thrombin (FIB) \ Light scattering	g/L	3.3	2.64-3.96
Sysmex CA 1500/6000/7000			
Siemens Multifibren U (FIB) \ Light scattering	g/L	3.58	2.86-4.3
(PT)血浆凝血酶原时间			
Werfen ACL series			
Werfen Hemosil PT Recombiplastin \ Light scattering	Seconds	11.7	9.36-14
Werfen ACL series			
Werfen Hemosil PT Recombiplastin \ Light scattering	INR	1.04	0.84-1.24
Werfen ACL series			
Werfen Hemosil PT Recombiplastin 2G \ Light scattering	Seconds	11.6	9.28-13.9
Werfen ACL series			
Werfen Hemosil PT Recombiplastin 2G \ Light scattering	INR	1.02	0.82-1.22
Werfen ACL series			
Werfen Hemosil PT-Fib HS Plus \ Light scattering	Seconds	15.9	12.7-19.1
Werfen ACL series			
Werfen Hemosil PT-Fib HS Plus \ Light scattering	INR	1.06	0.86-1.26
Stago STA series			
STA-Neoplastine CI Plus \ Magnetic Beads	Seconds	14.3	11.4-17.2
Stago STA series			
STA-Neoplastine CI Plus \ Magnetic Beads	INR	1.03	0.83-1.23
Sysmex CS 5100			
Siemens PT Innovin \ Light scattering	INR	1.13	0.91-1.35
Sysmex CS 5100			
Siemens Thromborel S (PT) \ Light scattering	Seconds	13.3	10.6-16
Sysmex CS 5100			
Siemens Thromborel S (PT) \ Light scattering	INR	1.06	0.84-1.28
(TT)凝血酶时间			
Werfen ACL series			
Werfen \ Light scattering	Seconds	14	11.2-16.8

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。