



REF	671-2	6×1mL	EXP	2026-04-07	LOT	4003486	更新日期:	2023-12
1 / 2								

本报告含以下项目的示范实验室数据

参数 (Parameters)	参数 (Parameters)	参数 (Parameters)
(APTT)活化部分凝血活酶时间	(AT-III)抗凝血酶III	(D-D(DDU)) D-二聚体(DDU)
(D-D(FEU)) D-二聚体(FEU)	(FDP)纤维蛋白(原)降解产物	(FIB)纤维蛋白原
(PT)血浆凝血酶原时间	(TT)凝血酶时间	

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。

REF	671-2	6×1mL	EXP	2026-04-07	LOT	4003486	更新日期：	2023-12
2 / 2								

批号：4003486			
项目\仪器\试剂方法	单位	均值	+ / - 2 SD
(APTT)活化部分凝血活酶时间			
Werfen ACL series Werfen Hemosil SynthAsil APTT \ Light scattering	Seconds	57.7	45.3-70.1
Werfen ACL series Werfen Hemosil APTT-SP \ Light scattering	Seconds	61.4	48.2-74.6
Stago STA series Stago \ Magnetic Beads	Seconds	63.2	49.6-76.8
Sysmex CA 1500/6000/7000 Siemens Dade Actin FS(APTT) \ Light scattering	Seconds	68.8	54-83.6
Sysmex CA 1500/6000/7000 Siemens Dade Actin(APTT) \ Light scattering	Seconds	87.4	68.6-106
Sysmex CA 1500/6000/7000 Siemens Pathrombin SL(APTT) \ Light scattering	Seconds	94.9	74.5-115
(AT-III)抗凝血酶III			
Werfen ACL series Werfen Hemosil Liquid Antithrombin (AT□) \ Nephelometry	%	48.6	38.9-58.3
Werfen ACL series Werfen IL Antithrombin III Chromogenic (AT□) \ Nephelometry	%	47.8	38.2-57.4
Sysmex CA 1500/6000/7000 Siemens Innovance Antithrombin (AT□) \ Nephelometry	%	48.5	38.8-58.2
Sysmex CA 1500/6000/7000 Siemens Berichrom Antithrombin III (AT□) \ Nephelometry	%	47	37.6-56.4
(D-D(DDU)) D-二聚体(DDU)			
Werfen ACL series Werfen Hemosil D-Dimer HS \ Nephelometry	mg/L	1	0.798-1.2
(D-D(FEU)) D-二聚体(FEU)			
Stago STA series Stago \ Nephelometry	mg/L	197	158-236
Sysmex CA 1500/6000/7000 Siemens Innovance Antithrombin (AT□) \ Nephelometry	mg/L	3.81	3.05-4.57
(FDP)纤维蛋白(原)降解产物			
Werfen ACL series SekiSui \ Nephelometry	mg/L	4.66	3.72-5.6
(FIB)纤维蛋白原			
Werfen ACL series Werfen Hemosil QFA Thrombin (FIB) \ Light scattering	g/L	2.1	1.71-2.49
Werfen ACL series Werfen Hemosil Fib-C \ Light scattering	g/L	1.95	1.58-2.32
Stago STA series Stago \ Magnetic Beads	g/L	2.19	1.76-2.62
Sysmex CA 1500/6000/7000 Siemens Thrombin (FIB) \ Light scattering	g/L	1.98	1.59-2.37
Sysmex CA 1500/6000/7000 Siemens Multifibren U (FIB) \ Light scattering	g/L	2.55	2.04-3.06
(PT)血浆凝血酶原时间			
Werfen ACL series Werfen Hemosil PT Recombiplastin \ Light scattering	Seconds	28.9	22.7-35.1
Werfen ACL series Werfen Hemosil PT Recombiplastin 2G \ Light scattering	Seconds	29.6	23.2-36
Werfen ACL series Werfen Hemosil PT-Fib HS Plus \ Light scattering	Seconds	37.6	29.5-45.7
Stago STA series Stago \ Magnetic Beads	Seconds	31.7	24.9-38.5
Sysmex CA 1500/6000/7000 Siemens PT Innovin \ Light scattering	Seconds	27	21.2-32.8
Sysmex CA 1500/6000/7000 Siemens Thromborel S (PT) \ Light scattering	Seconds	30.9	24.3-37.5
(TT)凝血酶时间			
Werfen ACL series SUNBIO \ Light scattering	Seconds	16.4	13.2-19.6
Sysmex CA 1500/6000/7000 Dedicated \ Light scattering	Seconds	16.3	13.1-19.5

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。